



76 Spruce Street, Cedarhurst, NY 11516
Tel: (516) 295-1121 Fax: (516) 295-2992
Lab@Implantlogic.com www.ImplantLogic.com

VIP Software in Windows Vista

The VIP software must be defined as “Administrator” in order to run in Windows VISTA

To make VIP “run as administrator” in Windows VISTA, please follow the below steps;

- 1- Right click on the VIP desktop icon
- 2- Click on "Properties"
- 3- Click on the "Compatibility" tab
- 4- Check on the box next to “Run this program as administrator”
- 5- Click APPLY and OK